

**Need Simplicity With
Implant Restorations?
Try The Provide®
Restorative System**



Introducing The Provide Restorative System

The Solution Designed To Provide Clinicians With Simpler Implant Restorations:

- **Simplified Restorative Protocol; As Easy As Crown And Bridge Procedures**
- **Allows Clinicians To Partner Together For A Simpler Surgical And Restorative Experience**
- **Easily Identifiable Color Coded Compatible Components**



Providing Clinicians One Solution At A Time With Provide Abutments

- **No Need To Determine Final Crown Margins During Time Of Surgery For Restorative Flexibility**
- **Certain® QuickSeat® Connection For Audible And Tactile Confirmation Of Abutment Seating**
- **Snap-on Impression Copings Eliminate The Need For Implant Level Impressions**

BIOMET 3i™
PROVIDING SOLUTIONS – ONE PATIENT AT A TIME™

A Simpler Surgical And Restorative Experience

With the increasing demands for dental implant therapy, the Provide® Abutment offers both surgical and restorative clinicians more options and greater flexibility to meet these demands and better serve patients. It allows them to partner together for a simpler surgical and restorative experience.

The implant is placed at bone level and not transgingivally, therefore the final crown margin is part of the abutment, not part of the implant. Provide Abutments offer clinicians four collar height options for unmatched flexibility. This means the final crown margin is determined by restorative indication, not surgical consequence.



Providing Surgical Flexibility

- **No Need To Determine Final Crown Margin At The Time Of Surgery**
- **No Need To Prepare Implant Collar Due To Tissue Recession**
- **Multiple Collar Heights (1, 2, 3 and 4mm)**
- **Use In A One Or Two-Stage Surgical Protocol**

Achieve Your Practice Growth Goals Faster With A Simpler Restorative Protocol



When clinicians want to reach their practice growth goals, using the Provide Abutment can be ideal. The simple restorative protocol is often easier than traditional crown and bridge procedures, which may allow for more implant referrals and greater adoption of implant therapy.

Because restorative results and component costs are more predictable, you may achieve your practice growth goals more consistently. The Provide Abutment simplifies implant dentistry and may allow for more predictable growth of that segment of your practice.

Easily Identifiable Color Coded Compatible Components

Provide® Component Selection Flowchart

	Blue	Blue	Yellow	Green	Yellow	Green
	 a = 4.1mm c = 4mm d = 4.8mm	 a = 4.1mm c = 5.5mm d = 4.8mm	 a = 5mm c = 4mm d = 6.5mm	 a = 6mm c = 4mm d = 6.5mm	 a = 5mm c = 5.5mm d = 6.5mm	 a = 6mm c = 5.5mm d = 6.5mm
Abutments	IPA4140 IPA4240 IPA4340 IPA4440	IPA4155 IPA4255 IPA4355 IPA4455	IPA5140 IPA5240 IPA5340 IPA5440	IPA6140 IPA6240 IPA6340 IPA6440	IPA5155 IPA5255 IPA5355 IPA5455	IPA6155 IPA6255 IPA6355 IPA6455
Screws	IUNIHG	IUNIHG	IUNIHG	IUNIHG	IUNIHG	IUNIHG
Protection Caps	PPC484	PPC485	PPC654	PPC654	PPC655	PPC655
Impression Copings	PIC484 Non-Prepared PIC484H Prepared	PIC485 Non-Prepared PIC485H Prepared	PIC654 Non-Prepared PIC654H Prepared	PIC654 Non-Prepared PIC654H Prepared	PIC655 Non-Prepared PIC655H Prepared	PIC655 Non-Prepared PIC655H Prepared
Laboratory Analogs	PAA484 Non-Prepared PAAP484 Prepared	PAA485 Non-Prepared PAAP485 Prepared	PAA654 Non-Prepared PAAP654 Prepared	PAA654 Non-Prepared PAAP654 Prepared	PAA655 Non-Prepared PAAP655 Prepared	PAA655 Non-Prepared PAAP655 Prepared
Waxing Sleeves	PWS48S Single Unit PWS48M Multi-Unit	PWS48S Single Unit PWS48M Multi-Unit	PWS65S Single Unit PWS65M Multi-Unit	PWS65S Single Unit PWS65M Multi-Unit	PWS65S Single Unit PWS65M Multi-Unit	PWS65S Single Unit PWS65M Multi-Unit
Temporary Cylinders	PUA48S Single Unit PUA48M Multi-Unit	PUA48S Single Unit PUA48M Multi-Unit	PUA65S Single Unit PUA65M Multi-Unit	PUA65S Single Unit PUA65M Multi-Unit	PUA65S Single Unit PUA65M Multi-Unit	PUA65S Single Unit PUA65M Multi-Unit

* b: collar height selection depends on tissue depth




Ordering Information



Provide® Abutments								
Platform Post Height	4mm				5.5mm			
Collar Height	1mm	2mm	3mm	4mm	1mm	2mm	3mm	4mm
4.1mm	IPA4140	IPA4240	IPA4340	IPA4440	IPA4155	IPA4255	IPA4355	IPA4455
5mm	IPA5140	IPA5240	IPA5340	IPA5440	IPA5155	IPA5255	IPA5355	IPA5455
6mm	IPA6140	IPA6240	IPA6340	IPA6440	IPA6155	IPA6255	IPA6355	IPA6455



Provide Kits								
4.1mm Platform			5mm Platform			6mm Platform		
Provide Abutment	Abutment Placement Kit*	Abutment Restorative Kit**	Provide Abutment	Abutment Placement Kit*	Abutment Restorative Kit**	Provide Abutment	Abutment Placement Kit*	Abutment Restorative Kit**
IPA4140	PAK4140	PRK484	IPA5140	PAK5140	PRK654	IPA6140	PAK6140	PRK654
IPA4240	PAK4240	PRK484	IPA5240	PAK5240	PRK654	IPA6240	PAK6240	PRK654
IPA4340	PAK4340	PRK484	IPA5340	PAK5340	PRK654	IPA6340	PAK6340	PRK654
IPA4440	PAK4440	PRK484	IPA5440	PAK5440	PRK654	IPA6440	PAK6440	PRK654
IPA4155	PAK4155	PRK485	IPA5155	PAK5155	PRK655	IPA6155	PAK6155	PRK655
IPA4255	PAK4255	PRK485	IPA5255	PAK5255	PRK655	IPA6255	PAK6255	PRK655
IPA4355	PAK4355	PRK485	IPA5355	PAK5355	PRK655	IPA6355	PAK6355	PRK655
IPA4455	PAK4455	PRK485	IPA5455	PAK5455	PRK655	IPA6455	PAK6455	PRK655

* Abutment Placement Kit includes the Provide Abutment, Protection Cap and Abutment Screw.
 ** Abutment Restorative Kit includes the Provide Impression Coping (non-prepared), Provide Impression Coping w/Holes (Prepared), Provide Waxing Sleeve (single unit and multi-unit) and Provide Abutment Analog (non-prepared) - The Provide Abutment Analog w/Pin (prepared) is NOT included.

Please Note: Provide Impression and Restorative Components are not compatible with the Straumann® ITI® Implant System.

Impression Copings					Laboratory Analogs					
	Non-Prepared Profiles		Prepared Post Profiles			Non-Prepared Profiles		Prepared Post Profiles		
	Post Height	4.8mm	6.5mm	4.8mm		6.5mm	Post Height	4.8mm	6.5mm	4.8mm
	4mm	PIC484	PIC654	PIC484H	PIC654H	4mm	PAA484	PAA654	PAAP48	PAAP65
	5.5mm	PIC485	PIC655	PIC485H	PIC655H	5.5mm	PAA485	PAA655	PAAP48	PAAP65

Protection Caps			Waxing Sleeves				
	Profiles		Profiles				
	Post Height	4.8mm	6.5mm	Post Height	4.8mm	6.5mm	
	4mm	PPC484	PPC654		PWS48S	PWS65S	Single Units
	5.5mm	PPC485	PPC655	5.5mm	PWS48	PWS65M	Multiple Units

Temporary Cylinders				Certain® Gold-Tite® Screw		
	Profiles				Abutment Screw	
	Post Height	4.8mm	6.5mm		IUNIHG	
	4mm, 5.5mm	PUA48S	PUA65S			Single Units
	4mm, 5.5mm	PUA48M	PUA65M			Multiple Units

Allows Clinicians To Partner Together For A Simpler Surgical And Restorative Experience


When you want quicker and easier implant restorations, you need the right product. The Provide® Abutment provides dental implant clinicians with easily identifiable, color coded compatible components. Provide Abutments feature the Certain® QuickSeat® Connection, with its audible and tactile “click” for verification that the abutment is correctly seated. The Provide Abutment’s snap-on impression components eliminate the need for an implant-level impression or titanium preparation. There is no need to pack cord, which may result in a restorative procedure that is as simple as traditional crown and bridge dentistry.



Providing Restorative Flexibility

- Snap-On Impression Components
- No Implant-Level Impression
- Variety Of Provisionalization Options (Provide Temporary Cylinder, Provide Protection Cap)
- Certain QuickSeat Connection
- Color Coded Platform, Post And Interfacing Components
- Two Post Heights (4mm, 5.5mm)

A Restorative Procedure That Challenges The Simplicity Of Traditional Crown And Bridge Dentistry

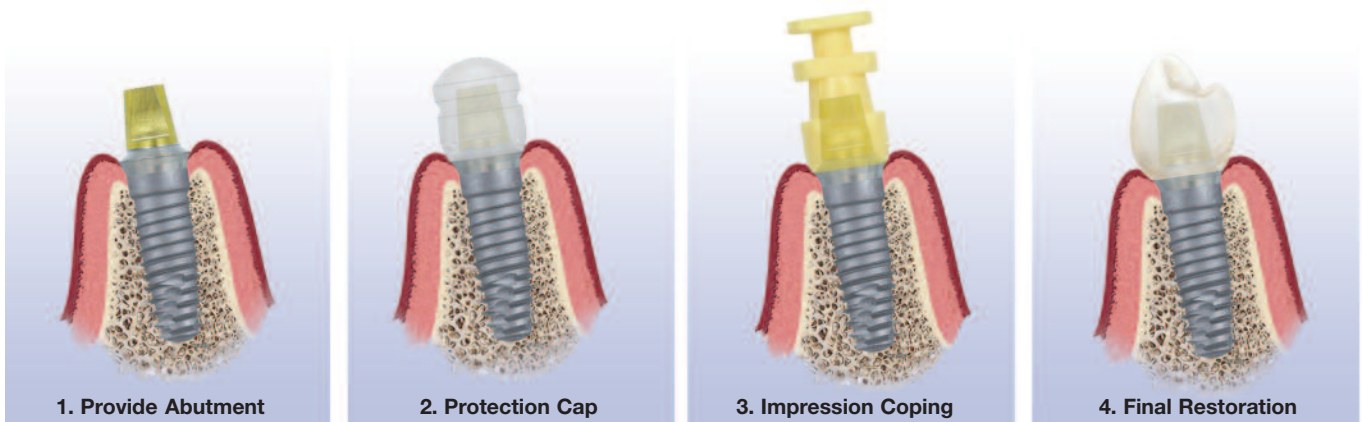

To Find Out How Simple Your Implant Restorations Could Be, Contact Your **BIOMET 3i** Sales Representative Today.

SUBSIDIARIES

AUSTRALIA

Phone: +61-2-9855-4444
Fax: +61-2-9888-9900

AUSTRIA

Phone: +43-(0)6235-200-45
Fax: +43-(0)6235-200-45-9

BELGIUM

Phone: +32-2-5410290
Fax: +32-2-5410291

BRAZIL

Phone: +55-11-5081-4405
Fax: +55-11-5081-7484

CANADA

Phone: +514-956-9843
Fax: +514-956-9844

FRANCE

Phone: +33-1-41054343
Fax: +33-1-41054340

GERMANY

Phone: +49-721-255177-10
Fax: +49-721-255177-73

IRELAND

Phone: +353-1-800-552-752
Fax: +44-1628-820182

JAPAN

Phone: +81-66868-3012
Fax: +81-66868-2444

KOREA

Phone: +82-2-5678-550
Fax: +82-2-5678-577

MEXICO

Phone: +52-55-2282-0120
Fax: +52-55-2282-0120 ext. 20

THE NETHERLANDS

Phone: +31-(0)78-629-2800
Fax: +31-(0)78-629-2801

NEW ZEALAND

Phone: +64-508-122-221
Fax: +64-508-133-331

NORDIC REGION

Phone: +46-40-17-6090
Fax: +46-40-17-6099

PORTUGAL

Phone: +34-902-34-34-31
Fax: +34-93-445-81-36
+351-21-000-16-75

SPAIN

Phone: +34-902-34-34-31
Fax: +34-93-445-81-36

SWITZERLAND

Phone: +41-(0)44-200-76-00
Fax: +41-(0)44-200-76-01

U.K.

Phone: +44-800-652-1233
Fax: +44-1628-820182

DISTRIBUTORS

ARGENTINA

Dentalmax, SA
Phone: +541-1482-71001
Fax: +541-1482-67373

CHILE

Cybel, SA
Phone: +56-2-2321883
Fax: +56-2-2330176

CHINA

Atek Inc.
Phone: +86-21-6329-1265
Fax: +86-21-6329-1620

COLOMBIA

3i Colombia
Phone: +571-612-9362
Fax: +571-620-6412

COSTA RICA

Implanteo S.A.
Phone: +506-234-9043
Fax: +506-224-7620

EL SALVADOR

Dentimer SA de CV
Phone: +503-263-6350
Fax: +503-263-6676

GREECE

Impladend Dental Implants, LLC
Phone: +30-2310-501-651
Fax: +30-2310-862-090

ISRAEL

H.A. Systems
Phone: +972-3-6138777
Fax: +972-3-6138778

HONG KONG

Ositek Inc., Ltd.
Phone: +852-8121-6601
Fax: +852-3747-3754

ITALY

Biomax spa
Phone: +39-0444-913410
Fax: +39-0444-913695

MIDDLE EAST

3i MENA s.a.l.
Middle East And North Africa
Phone: +961-1-694000
Fax: +961-1-694222

PARAGUAY

Andres H. Arce y Cia SRL
Phone: +595-21-208185
Fax: +595-21-496291

POLAND

Dental Depot Wasio
Phone: +48 71 335 70 71
Fax: +48 71 335 70 90

RUSSIA

Com-Dental
Phone: +7-495-580-3080
Fax: +7-495-580-3081

SINGAPORE

Asia Implant Support & Services
Phone: +65-6223-2229
Fax: +65-6220-3538

SOUTH AFRICA

Selective Aesthetics
Phone: +27-11-991-7068
Fax: +27-11-672-1391

TAIWAN

Kuo Hwa Dental Suppliers Co., Ltd.
Phone: +886-2-2226-1770
Fax: +886-2-2226-8747

THAILAND

3i (Thailand) Co., LTD.
Phone: +662-252-6685
Fax: +662-252-6686

TURKEY

Tamer Med. A.Ş.-Turkey
Phone: +90-212-465-3352
Fax: +90-212-465-3502

UKRAINE

Victoria Premium
Phone: +38-044-331-10-09
Fax: +38-044-331-10-70

URUGUAY

Pro3implant S.R.L.
Phone: +598-2-4034163
Fax: +598-2-4034163

VIETNAM

D.O.E.
(Dentistry Of Excellence Vietnam)
Phone: +84 89 25 37 03
Fax: +84 89 25 37 00

Certain,
Gold-Tite,
Provide and
QuickSeat are
registered
trademarks and
Providing Solutions
- One Patient At A

Time and design are trademarks of BIOMET 3i LLC.
BIOMET 3i and design are trademarks of BIOMET, Inc.
Straumann is a federally registered trademark of
Straumann Holding AG Corp, Switzerland. ITI is a federally
registered trademark of Straumann Holding AG Corp,
Switzerland ©2010 BIOMET 3i LLC. All rights reserved.



REV D 12/10

***Not Available In All Markets. Please Consult Your Local **BIOMET 3i** Sales Representative For Availability**

To Receive Information About **BIOMET 3i's Products, Services And Events By Email,
Please Visit Our Website At www.biomet3i.com**

BIOMET 3i

4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454
Outside The U.S.: +1-561-776-6700
Fax: +1-561-776-1272
www.biomet3i.com

EC REP

BIOMET 3i

Europe, Middle East & Africa
WTC Almeda Park, Ed. 1, Planta 1ª
Pl. de la Pau, s/n
08940, Cornellà de Llobregat
(Barcelona) Spain
Phone: +34-93-470-55-00
Fax: +34-93-371-78-49

BIOMET 3iTM
PROVIDING SOLUTIONS - ONE PATIENT AT A TIMETM